



Quick Start Guide

For BlueCHIP Certified workstations, only follow steps marked with



1 Connect Batteries (Li-Nano Only)



A. Remove cover



B. Connect batteries



C. Replace cover and tighten screws

2 Setup Monitor



A. Loosen screws and remove union cover



B. Loosen screws and remove monitor neck cover



C. Remove work surface top



D. Route monitor cables through tech tray





E. Connect monitor



F. Adjust monitor tension, if needed (see Monitor Arm Adjustment in operator manual for details)

3 Setup Technology

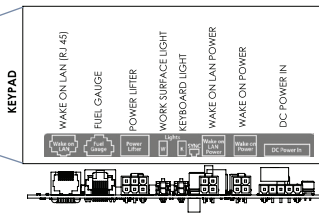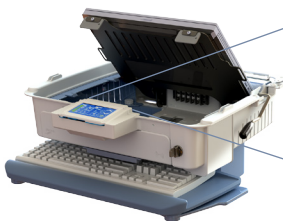

A. Select Wake on Power or Wake on LAN (see computer's user guide)

B. Connect applicable cables

C. Push Sync button

4 Charge Batteries



A. Plug into working outlet



B. Turn on power supply



C. Verify lights on keypad

For detailed set-up instructions, please refer to the AccessPoint user guide.

www.touchpointmed.com
All rights reserved. Touchpoint Medical logo is trademark of TouchPoint Medical.

L07-093 Rev. C 02/2017

